Physio 5

	USER MANUAL..... 1
	GUIDE DE L'UTILISATEUR... 18
	BEDIENUNGSANLEITUNG ... 36
	MANUALE UTENTE..... 54
	MANUAL DEL USUARIO 72

Manufacturer:
DJO, LLC
1430 Decision Street
Vista, CA 92081 U.S.A.

EU Authorised Representative
MDSS GmbH
Schiffgraben 41
30175 Hannover
Germany

0473

© 2010 DJO, LLC

Cefar-Compex® is a registered trademark of DJO, LLC.

CONTENTS

I INTENDED USE	2
II WARNINGS	2
1 - Counter-indications	2
2 - Safety measures	2
III PRESENTATION	4
1 - Warranty	4
2 - Maintenance	4
3 - Storage and transportation conditions	4
4 - Usage conditions	4
5 - Elimination	4
6 - Standards	4
7 - Standardized symbols	5
8 - Technical specifications	5
A) General information	5
B) Neurostimulation	5
C) Information related to electromagnetic compatibility (EMC)	5
IV USAGE GUIDELINES	6
1 - Placement of electrodes	6
2 - Body position	6
3 - Adjusting stimulation energies	6
4 - Level progression	6
V USING THE STIMULATOR	7
1 - Description of stimulator	7
2 - Connections	8
A) Connecting the cables	8
B) Connecting the motor point pen	8
C) Charging	8
3 - Preliminary settings	9
4 - Neurostimulation programmes	9
A) Choosing a type of treatment	9
B) Choosing a programme category	9
C) Choosing a programme	10
D) Personalising a programme	10
E) Programming mode	11
F) Stimulation mode	12
5 - Direct currents	13
A) Iontophoresis	13
B) Hyperhidrosis	13
6 - Denervated programme	14
7 - Battery status and recharging	15
A) Battery status	15
B) Recharging	15
8 - Troubleshooting	16
VI LIST OF PROGRAMMES	17

I - INTENDED USE

The Physio5 is a stimulator intended to be used by health professionals to provide electrical muscle stimulation, iontophoresis and denervated treatments.

II - WARNINGS

1 - Counter-indications

- Do not treat persons with implanted electronic equipment, such as pacemakers and intracardiac defibrillators.
- Avoid electrode placement at the front and side areas of the neck.
- Avoid electrode placement on the head.
- Do not treat pregnant women (unless approved by referring gynaecologist).
- Do not use thoracic stimulation on patients with cardiac arrhythmia.
- Avoid lower limb stimulation in case of venous thrombosis or severe arterial obstruction (ischemia).
- Avoid abdominal stimulation if you have abdominal or inguinal hernia.

Important!

- Read this manual and the accompanying Practical Guide carefully and completely.

2 - Safety measures

Important points to take into account when using Physio5:

- Treatment should never be applied near the area of an implant, such as cochlear, pacemakers, skeletal or electrical.
- Do not apply stimulation in the vicinity of metal. Remove jewellery, body piercings, buckles or any other removable metallic product or device in the area of stimulation.
- Do not use the stimulator in water or humid environments (saunas, hydrotherapy pools, etc.).
- If pregnant or menstruating, do not place electrodes directly over the uterus or connect pairs of electrodes across the abdomen.
- Observe caution if treating patients with sensitivity problems or who are unable to let you know if they feel any discomfort, however slight.
- Observe caution when stimulating in the immediate vicinity of cellular phones that are switched on, since this may affect the output power of the stimulator.
- Do not use the stimulator on a patient who is close or connected to a high-frequency surgical instrument as this could cause skin irritation or burns under the electrodes.
- Do not use the stimulator within one metre of short wave or microwave devices as this could alter the currents generated by the stimulator. If you are in any doubt as to the use of the Physio5 close to another medical device, seek advice from the manufacturer of the latter or from your doctor.
- Observe caution when using electrotherapy at the same time as the patient is connected to monitoring equipment with body worn electrodes. The stimulation might interfere with the signals to the monitoring equipment.

- Only use stimulation cables, rechargeable battery units and charger supplied by CefarCompex.
 - Always turn off the stimulator or make sure that the intensity for each channel is 0 before disconnecting cables from stimulator or electrodes.
 - Never connect the stimulation cables to an external power source due to risk of electric shock.
 - Do not use the Physio5 stimulator or charger if any of the elements are damaged (case, cables, etc.) or if the battery compartment is open. There is a risk of electric shock.
 - Unplug the charger immediately if your Physio5 emits a continuous beeping sound or if you notice any abnormal heat, smell, or smoke from either the charger or the stimulator.
 - Never recharge the battery in a confined space (carrying case, etc.) as this could cause a fire or electric shock.
 - Keep the Physio5 and its accessories out of the reach of children.
 - Keep the stimulator, battery compartment and charger clean.
 - Do not allow any foreign bodies (soil, water, metal, etc.) to penetrate the Physio5, the battery compartment and the charger.
 - Sudden temperature changes can cause condensation to build up inside the stimulator. To prevent this, allow it to reach ambient temperature before use.
 - Do not use your Physio5 while driving or operating machinery.
 - Do not apply stimulation while sleeping;
 - The long-term effects of electrical stimulation are unknown;
 - TENS is a symptomatic treatment and, as such, suppresses the sensation of pain that would otherwise serve as a protective mechanism. TENS programmes have no curative value.
- 
- Precautions when using the electrodes and the motor point pen:**
- Do not attempt to place electrodes on any part of the body not directly visible without assistance.
 - Only use electrodes and pen supplied by CefarCompex. Other electrodes and pens may have electrical properties that are incompatible with the Physio5 stimulator.
 - Only place electrodes on healthy skin and never on skin lesions of any kind (wounds, swelling, burns, irritation, eczema, etc.).
 - Always turn off the stimulator or make sure that the intensity for each channel is 0 before moving or removing the electrodes.
 - Do not place the electrodes or the pen in water.
 - Do not apply solvents of any kind to the electrodes or pen.
 - Do not use other electrodes than those provided by CefarCompex as there will be a risk of suffering a burn injury. Caution should always be exercised with current densities > 2mA/cm².
 - For optimal results, wash, clean and dry the skin before attaching the electrodes.
 - Attach the electrodes so the entire surface is in contact with the skin.
 - For hygiene reasons, each patient must have his own set of electrodes. Never use the same electrodes on different patients.
 - Some people with very sensitive skin may experience redness under the electrodes after a session. This redness is generally harmless and normally disappears within 10 to 20 minutes. Do not start another stimulation session in the same area, as long as the redness is still visible.
 - When the electrodes lose their adhesiveness and do not stick properly to the skin, the stimulation can feel unpleasant. Be sure that the electrodes are in good condition, otherwise they have to be replaced.
 - Never use a set of adhesive electrodes for more than 15 sessions as their bonding power deteriorates over time and optimal contact is very important for both user comfort and efficacy.
 - Clean and disinfect the tip of the motor point pen that comes in contact with the skin before each use.

III PRESENTATION

1 - Warranty

This warranty is valid only if accompanied by proof of purchase. Your legal rights are not affected by this warranty.

Your CefarCompex Physio5 stimulator is guaranteed for a period of 2 years from the date of purchase. The warranty covers the stimulator (material and workmanship) but not the battery unit, cables, charger, electrodes or motor point pen. All defects arising from substandard material or workmanship are covered. The warranty does not cover damage resulting from impact, accident, improper use, insufficient protection against damp, immersion in water, or repair work conducted by unauthorized personnel.

2 - Maintenance

Clean with a soft cloth and an alcohol-based, solvent-free cleaning product. Use only a minimum amount of liquid when cleaning the unit.

Do not disassemble the stimulator or charger as these all contain high-voltage components that could cause electric shock. This may only be done by persons or repair services authorized by CefarCompex.

Your stimulator does not require calibration. If your stimulator contains parts that seem worn or defective, please contact your nearest CefarCompex customer service centre.

3 - Storage and transportation conditions

The Physio5 contains rechargeable batteries and so the storage and transportation conditions must not exceed the following figures:

- Storage and transportation temperature: -20°C to 45°C
- Maximum relative humidity: 75%
- Atmospheric pressure: 700 hPa to 1060 hPa

4 - Usage conditions

- Temperature: 0°C to 40°C
- Relative humidity: 30% to 75%
- Atmospheric pressure: 700 hPa to 1060 hPa.
- Do not use in an explosion risk area.

5 - Elimination

Batteries must be disposed in compliance with relevant national regulatory requirements.

Any products bearing the WEEE marking (a crossed-out wheeled bin) must be separated from ordinary household waste and sent to special collection facilities for recycling and recovery.

6 - Standards

The CefarCompex Physio5 is designed for use by health professionals and as such complies with relevant medical standards and regulatory requirements.

To guarantee your safety, the CefarCompex Physio5 has been designed, manufactured, and distributed in compliance with the requirements of the European Directive 93/42/EEC on medical devices.

The CefarCompex Physio5 also complies with the IEC 60601-1 standard on general safety requirements for medical electrical devices, the IEC 60601-1-2 standard on electromagnetic compatibility, and the IEC 60601-2-10 standard on particular safety requirements for nerve and muscle stimulators.

Current international standards require that a warning be given concerning the application of electrodes to the thorax (increased risk of cardiac fibrillation). The CefarCompex Physio5 also complies with Directive 2002/96/EEC on waste electrical and electronic equipment (WEEE).

7 - Standardized symbols

Read the operating instructions before use.

The CefarCompex Physio5 is a class II device with internal electric power and type BF applied parts.

Waste electrical and electronic equipment (WEEE) marking according to EN 50419 standard.

8 - Technical specifications

A) General information

Battery 941213: Nickel metal hydride (NiMH) rechargeable battery (4.8 V, 2000 mAh).

Battery chargers: Only battery chargers bearing the part number 68301X can be used to recharge the batteries supplied with the Physio5 stimulator.

B) Neurostimulation

All electrical specifications are given for an impedance of 500-1000 Ohms per channel.

Channels: Four individually adjustable channels that are electrically isolated from each other and earthed.

Impulse shape: Constant rectangular current with pulse compensation to eliminate any direct current component to prevent residual polarization at skin level.

Maximum pulse intensity: 120 mA.
Pulse intensity increments: Manual adjustment of stimulation intensity: 0-999 (energy units) - Minimum increment: 0.5 mA.

Pulse width: 30 to 400 microseconds
Maximum electrical charge per pulse: 96 microcoulombs (2 x 48 µC, compensated).

Standard pulse ramp-up time: 3 microseconds (20%-80% of maximum current).

Pulse frequency: 1-150 Hz.

C) Information related to electromagnetic compatibility (EMC)

The Physio5 is designed to be used in typical domestic or clinical environments and is approved according to the EMC safety standard of IEC 60601-1-2.

This device emits very low levels in the radio frequency (RF) interval and is therefore not likely to cause any interference with nearby electronic equipment (radios, computers telephones etc.).

The Physio5 is designed to withstand foreseeable disturbances originating from electrostatic discharges, mains supply magnetic fields or radio frequency transmitters.

Despite this, it is not possible to guarantee that the stimulator is not affected by strong RF (radio frequency) fields emitting from for example mobile phones.

For more detailed information regarding electromagnetic emission and immunity, please contact CefarCompex.

IV USAGE GUIDELINES

1 - Placement of electrodes

For optimal results, use the electrode positions recommended by CefarCompex. For the placement of electrodes, refer to the electrode placement illustration. It is crucial to choose the right size of the electrodes and correctly position these on the muscle group you want to stimulate to ensure the efficacy of the treatment. To do this, always use the sizes and positions shown in the pictograms unless otherwise required by medical needs. Where necessary, look for the best possible position by slowly moving the electrode over the muscle until you find the point that will produce the best contraction (the motorpoint) or the most comfort for the patient.

CefarCompex disclaims all responsibility for consequences arising from electrodes placed in other positions.

2 - Body position

Refer to the pictures of the electrodes placement to determine the stimulation position for your patient depending on the placement of the electrodes and the chosen programme. This position will vary depending on the position of the electrodes, the muscle group you wish to stimulate, and the programme you are using. For programmes that do not involve muscle contractions (i.e.: Pain relief programmes), make sure that the patient is as comfortable as possible.

3 - Adjusting stimulation energies

The number of fibres recruited in a stimulated muscle depends on the stimulation energies used. It is therefore extremely important to use maximum stimulation energies where possible in order to recruit the maximum number of fibres. If the stimulation energy is too low, too few fibres will be obliged to work and the improvement in muscle quality will be negligible.

It takes at least 3 sessions to reach this maximum energy level. The first few sessions serve to introduce progressive increments in stimulation energy (and muscle contraction) levels and to help the patient to become used to the method.

After an initial warm-up period, which should produce very clear muscle twitches, increase the stimulation energies gradually, contraction by contraction, for the duration of the session.

4 - Level progression

Generally speaking, it is not advisable to try to reach maximum stimulation levels too quickly. The different levels in each stimulation programme are designed to re-educate the muscle gradually.

We recommend you to prioritize the stimulation energies at each level to recruit as many fibres as possible, before moving on to the next level. Also seek to progressively increase energy levels from session to session.


For maintenance purposes, it is a good idea to hold one session per week with the energy level you used in the previous cycle.

V USING THE STIMULATOR

You are strongly advised to read the counter-indications and safety measures described at the beginning of this manual (chapter 2 “Warnings”) prior to using your stimulator.

1 - Description of stimulator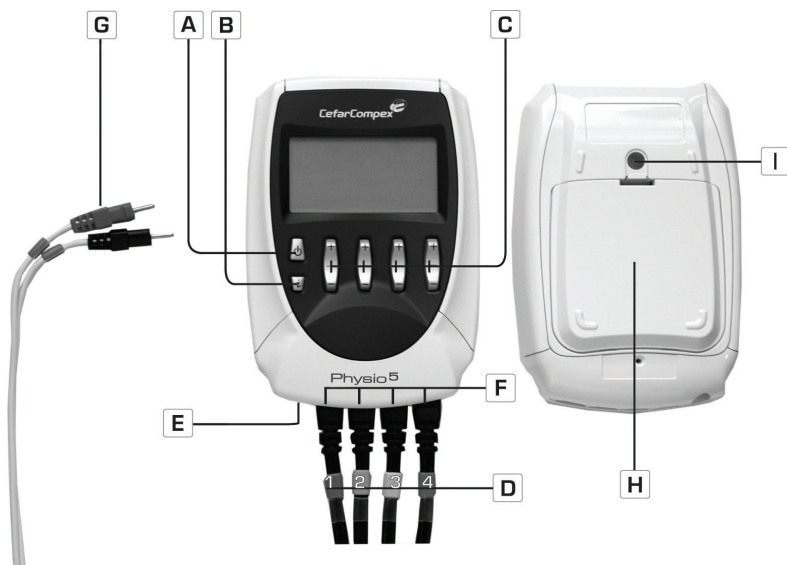

- A - On/Off button
- B - i button. This is used to:
 - Increase stimulation energies in several channels simultaneously.
 - Access the Top 5 menu (showing 5 most recently used programmes)
 - Access parameters info menu for personalised programmes.
- C - +/- buttons for 4 stimulation channels
- D - Sockets for 4 stimulation cables
- E - Battery charger socket

- F - Stimulation cables Channel 1 = blue
Channel 2 = green Channel 3 = yellow
Channel 4 = red
- G - Stimulation cable fitted with pin connector
- H - Rechargeable battery compartment
- I - Belt clip socket

2 - Connections

A) Connecting the cables


The stimulator cables plug into the 4 sockets on the base of the device. All 4 cables can be connected simultaneously. Both the +/- buttons and the cables are colour-coded to simplify use and facilitate identification of the different channels.

The kit includes four 2,1 mm pin-connector cables and 8 pin to snap converters.

B) Connecting the motor point pen

Never use the motor point pen for any purposes other than locating the motor point of the muscle. Follow the instructions in this manual. Clean and disinfect the tip of the motor point pen that comes in contact with the skin before each use. Apply a small amount of gel to the skin when looking for the motor point to enhance patient comfort.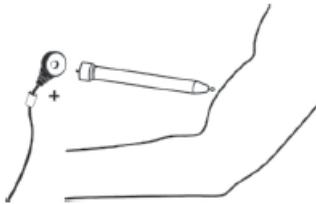

Refer to the picture above to see how to connect the pen.

Connect the tip of the motor point pen (preferably to the red connector). The other connector must be connected to an electrode already on the muscle to be stimulated.

C) Charging

The Physio5 is a portable muscle stimulator powered by a rechargeable battery unit.

Recharging


To recharge the Physio5, first disconnect the electrode cables from the device, then plug the charger to a wall socket and finally connect the stimulator to the charger.

The charge menu illustrated below appears automatically.

1a: The stimulator has been charged for 7 minutes and 27 seconds (a full charge may take 2-2,5 hours with the rapid charger delivered with your device).

1b: Charging is in progress. When charging is completed, total charge duration flashes and the battery symbol is completely full. The stimulator switches off automatically as soon as you disconnect the charger.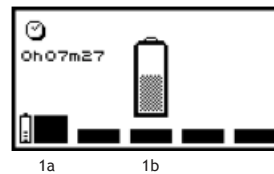

3 - Preliminary settings

A) Language, contrast, volume

When you start the stimulator for the first time, you will need to choose the interface language you wish to use from the options screen. See below for instructions on how to proceed. There are a range of settings that you can adjust to suit your needs (interface language, display contrast, backlighting, volume). To change any of these settings, bring up the options screen by holding the On/Off button on the front of the stimulator for a few seconds.

3b: Use the channel 1 +/- button to choose the language you wish to use.

3c: Use the channel 2 +/- button to adjust the contrast of the screen.

3d: Use the channel 3 +/- button to adjust the volume.

3e: Use the channel 4 +/- button to adjust the backlighting.

On: Backlighting always on.

Off: Backlighting always off.

Auto: Backlighting activated whenever a button is pressed.


3a: Use the On/Off button to confirm and save your choices. The settings will be applied immediately.

4 - Neurostimulation programmes

To start the stimulator, press the On/Off button. To choose a programme, you must first choose a type of treatment and a programme category.

NOTE: There is a table summarizing the different programmes and their functions at the end of this manual.

A) Choosing a type of treatment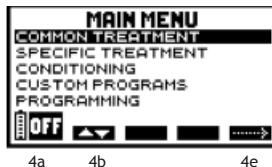


4a: Press the On/Off button to turn off the unit.

4b: Use the channel 1 +/- button to choose another type of treatment (i.e.: Specific Treatment or Conditioning).

4e: Press the channel 4 +/- button to confirm your choice and access the programme category selection screen. Press the i button to access the Top 5 menu.

B) Choosing a programme category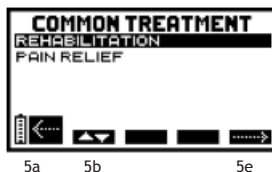


5a: Press the On/Off button to return to the previous screen.

5b: Use the channel 1 +/- button to choose a category.

5e: Press the channel 4 +/- button to confirm your choice and access the programme selection screen. Press the i button to access the Top 5 menu.

C) Choosing a programme

NOTE: Consult our Practical Guide for help in choosing a programme. Once you choose a category, the screen will display a list of available programmes.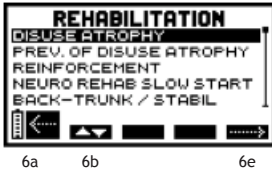


6a: Press the On/Off button to return to the previous screen.

6b: Use the channel 1 +/- button to choose the programme you want.

6e: Press the channel 4 +/- button to confirm your choice. Certain programmes start right away while others allow you to specify additional options.

TOP 5

To access the Top 5 menu, press the **i** button before selecting a programme. The menu can be accessed from the type of treatment screen (fig. 4), the programme category screen (fig. 5), or the programme list screen (fig. 6).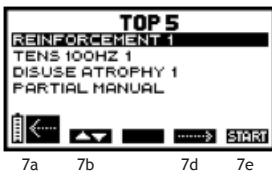

7a: Press the On/Off button to return to the previous screen.

7b: Use the channel 1 +/- button to choose the programme you want.


7d: Use the channel 3 +/- button to personalize the programme.

7e: Press the channel 4 +/- button to start the programme.

D) Personalising a programme

Not all programmes have an option screen.

In certain programmes, you will need to choose the muscle group you want to stimulate. The target area is shown in black on the figure above channel 1.

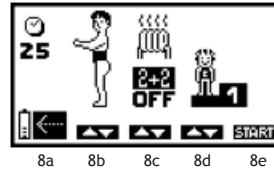8a: Press the On/Off button to return to the previous screen.

8b: Use the channel 1 +/- button to choose the desired treatment area. You can choose from 7 areas.

8c: Press the channel 2 + button to delete the warming-up sequence and 2 - to choose if you want to combine with any of the 2+2 programmes on channel 3 and 4 (see explanation hereunder).

8d: Use the channel 3 +/- button to choose the programme level.

8e: Use the channel 4 +/- button to confirm your choices and start the programme.

Choosing the 2+2 function

You have the possibility to choose the 2+2 function which means that you can choose one programme for channels 1 and 2, then select one programme from the 2+2 programme list for channels 3 and 4. So you can treat two different body parts at the same time or combine two different programmes on the same body part.

NOTE: The timer for channel 1+2 will control the total session time. This means that the total time for channel 3+4 can not exceed the session time for channel 1+2. The programme for channel 3+4 is always in one phase.

If a preset programme uses more than two channels, it's not available for the 2+2 mode.

E) Programming mode

In the programming mode you could create a custom programme and select your own parameters. Then name your programme and save it in the custom programmes category.

Go to the programming category.

Select NMES or TENS

Select the stimulation form

In order to personalize the parameters:


9a: Press the On/Off button to return to the previous screen.

9b: Use the channel 1 +/- button to move the cursor between the different settings.

9c: Use the channel 2 +/- button to increase or decrease the setting values.

9e: Use the channel 4 +/- button to save the programme.

Saving programme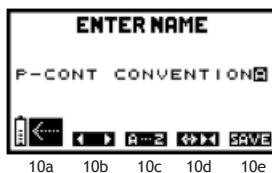

10a: Press the On/Off button to return to the previous screen.

10b: Use the channel 1 +/- button to move the cursor in the programme name.

10c: Use the channel 2 +/- button to write letters and figures.

10d: Use the channel 3 +/- button to maximize or minimize the space.


10e: Use the channel 4 +/- button to save the programme.

NOTE: You can save maximum 10 custom programmes.

NOTE: Custom programmes can be combined with a programme from the 2+2 list by adding and selecting the 2+2 option. This programme will consist of one programme on channel 1+2 (P1) and one programme for channel 3+4 (P2) (please see section "Choosing the 2+2 function").

F) Stimulation mode

When you start a programme, you will be prompted to increase the stimulation energies. This is the key to the success of any treatment.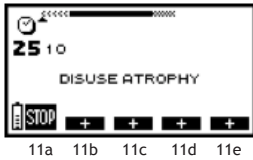

11a: Press the On/Off button to interrupt the programme momentarily.

11 b c d e: The unit will beep and the symbols for the 4 channels will alternate between + and 000. The energy level for the 4 channels is at 0. To start the programme, you will need to increase the energy levels in the channels you are going to use. To do this, use the corresponding + buttons. The different energies reached during the contraction phase are shown by a series of black bar graphs. Active rest phase energies are shown by hatched bar graphs.

NOTE: If you want to increase the energy levels in all 4 channels simultaneously, press the i button. Press it twice to increase the levels in the first 3 channels, and 3 times to increase the levels in the first 2 channels. Interdependent channels will be highlighted in white against a black background.

NOTE: Active rest phase stimulation energies are automatically set at 50% of contraction intensities but can be modified during the rest phase. Once modified, they will be totally independent of the contraction intensities.

NOTE: If your Physio5 emits a beeping sound and the + symbols under the active channels begin to flash, the stimulator is suggesting you to increase the level of the stimulation energies. If you are working at the patient's maximum tolerance level, simply ignore this message.

If you start a programme with the 2+2 function the following screen will be displayed.


The horizontal bars on top of the display shows the total duration and different phases of the programmes. Left bar for P1 and right bar for P2.

P1 shows the programme chosen for channel 1 and 2.

P2 shows the programme chosen for channel 3 and 4.


Pause mode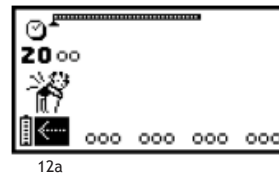

12a: Press the On/Off button to interrupt the programme momentarily. To restart it, simply press the + button of any channel. The session will resume at 80% of the energy levels that were being used prior to the interruption.

End of programme

At the end of each session, a small flag will be displayed on the screen and a short melody will be played.

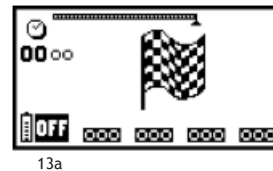13a: To turn off the stimulator, press the ON/OFF button.

5 - Direct Currents

Never use direct currents on patients with osteosynthesis devices or other metal implants. Iontophoresis and Hyperhidrosis programmes use Direct currents. Do not use these currents in the chest region.

A) Iontophoresis

Read the chapter “Iontophoresis” in the practical guide to become fully familiar with this programme before using it.

NOTE: Use the coloured electrodes (red, green, yellow) supplied with the stimulator whenever you use an Iontophoresis programme.

The Iontophoresis programme is in the direct current category in the specific type of treatment.

14a 14b 14e

14a: Press the On/Off button to return to the previous screen.

14b: Use the channel 1 +/- button to choose the category you want.

14e: Press the channel 4 +/- button to confirm your choice and access the programme selection screen.

15a 15b 15e

15a: Press the On/Off button to return to the previous screen.

15b: Use the channel 1 +/- button to choose a programme.

15e: Press the channel 4 +/- button to confirm your choice.

When working with an Iontophoresis programme, you can choose the number of channels and size of electrodes you want to use. There are 3 electrode sizes (red, green, and yellow).

You can also edit the default parameters - Duration, Electrical density -

recommended by CefarComplex.

16a 16b 16c 16e


16a: Press the On/Off button to return to the previous screen.

16b: Use the channel 1 +/- button to select the parameters you wish to edit: channel to use, density, and duration of session. In the above example, only channel 1 (green electrodes) is active.

The electrical density is 0.05 mA and the session duration is 6 minutes.

16c: Use the channel 2 +/- button to choose the electrode size (colour) you wish to use for each channel and/or to modify the default values (density, duration).

16e: Press the channel 4 +/- button to confirm your choices and access a confirmation screen.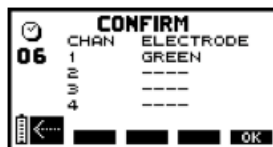

17a 17e

17a: Press the On/Off button to return to the previous screen

17e: Use the channel 4 +/- button to confirm your choices and start the programme.

NOTE: Current intensity is automatically calculated according to the size of electrode used. Electrode size is determined by colour. Make sure you choose the correct colour before running the programme.

Press the i button from the stimulation screen or the end of programme screen to access the programme parameters.


B) Hyperhidrosis

Read the chapter “Hyperhidrosis” in the Practical Guide to become fully familiar with this programme before using it.

NOTE: The Hyperhidrosis programme should always be run on channel 1 with

the large (red) Iontophoresis electrodes supplied with the stimulator.


The Hyperhidrosis programme is in the Direct Current category of the Specific Treatment type.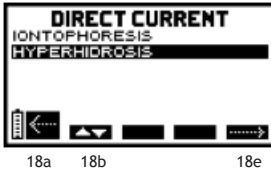

18a: Press the On/Off button to return to the previous screen.

18b: Use the channel 1 +/- button to choose a programme.

18e: Press the channel 4 +/- button to confirm your choice.

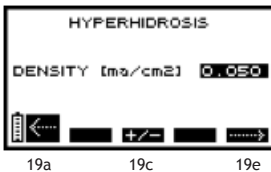19a: Press the On/Off button to return to the previous screen.

19c: Use the channel 2 +/- button to change the default electrical density value.

19e: Press the channel 4 +/- button to confirm your choices and access a confirmation screen.

Use the i button to access the electrodes placement pictogram.

The Hyperhidrosis programme lasts for 12 minutes and can only be run on channel 1. Only use the red electrodes with this programme. Other electrodes will cause current intensity calculation errors.

20a: Press the On/Off button to return to the previous screen.

20e: Press the channel 4 +/- button to confirm your choices and start the programme.

Use the i button to access the electrode

placement pictogram.

NOTE: Press the i button from the stimulation screen or the end of programme screen to access the programme parameters.

6 - Denervated programme

The Denervated programmes are part of the Specific Treatment. Do not use these programmes in the chest region.

Physio5 Denervated programmes are not contraindicated in patients with osteosynthesis devices or other metal implants

Denervated programmes should only be run with 2.1 mm pin-connector cables and carbon electrodes (black electrodes supplied with stimulator). For optimal results, apply conductor gel to the electrodes before use. Do not use snap-connector cables or selfadhesive electrodes for Denervated programmes.

There are 2 types of Denervated treatments.

21a: Press the On/Off button to return to the previous screen.


21b: Use the channel 1 +/- button to choose the programme you want.

21e: Press the channel 4 +/- button to confirm your choice.

Total manual

Full denervation programme. This programme works with rectangular pulses and allows the following parameters to be modified:

- Pulse width
- Period
- Programme duration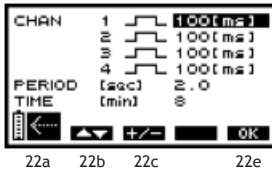

22a: Press the On/Off button to return to the previous screen.

22b: Use the channel 1 +/- button to select the parameter you wish to edit: pulse width for each channel, period, or programme duration.


22c: Use the channel 2 +/- button to edit the value of the parameter you have selected.

22e: Press the channel 4 +/- button to confirm your choice and start the treatment.

Partial manual

Partial denervation with choice of stimulation parameters to define the desired ramp.

To use a ramp of your choice, first select the pulse width option in the corresponding channel. Confirm your choice and increase the stimulation intensity to the desired level.

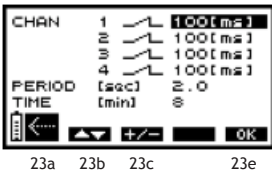23a: Press the On/Off button to return to the previous screen.

23b: Use the channel 1 +/- button to select the parameter you wish to edit: pulse width for each channel, period, or programme duration.

23c: Use the channel 2 +/- button to edit the value of the parameter you have selected.

23e: Press the channel 4 +/- button to confirm your choice and start the treatment.

7 - Battery status and recharging

Never recharge the stimulator without first disconnecting the stimulation cables.

Never recharge the battery with a charger other than that supplied by CefarComplex.

The Physio5 is powered by a rechargeable battery unit. The battery's duration varies according to the programmes and stimulation energies used. We strongly recommend you to fully charge the battery before using it for the first time as this will extend battery duration and life time. If you do not use your stimulator for long periods of time, remember to recharge the battery regularly or to store it outside the stimulator.

A) Battery status

The status of the battery is indicated by a battery icon on the screen. If the battery icon does not contain any lines, this means that power is running low. Do not start a session before recharging the unit.

If the START symbol normally displayed above the channel 4 +/- button is not visible and the battery icon is flashing, this means that the battery is completely discharged. The stimulator can no longer be used. Recharge it immediately.


B) Recharging

See chapter V, section 2, letter C "Charging".

8 - Troubleshooting

Electrode or cable error

The Physio5 will beep and alternately display a pair of electrodes and an arrow pointing to the channel where the problem was detected. In the example below, the stimulator has detected an error in channel 2.

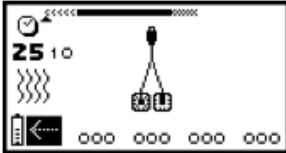The electrode error message can mean that:

- There are no electrodes connected to this channel.
- The electrodes are old, worn, and/or the contact is poor: try using new electrodes.
- The electrode cable is faulty: try connecting it to another channel. If the problem persists, replace the cable.

If for any reason your device seems to present a malfunction other than those presented above, please contact CefarCompex customer service.

VI LIST OF PROGRAMS - Physio5

ENG

CefarCompex Physio5 - USER MANUAL

COMMON TREATMENT	
PROGRAMMES	Optional 2+2
REHABILITATION	
DISUSE ATROPHY	x
PREV. OF DISUSE ATROPHY	x
REINFORCEMENT	x
NEURO REHAB SLOW START	x
BACK-TRUNK/ STABIL	x
ATROPHY (MOD FREQ)	x
FORCE (MOD FREQ)	x
INCREASE CIRCULATION	x
MUSCLE LESION	x
MOTOR POINT	
PAIN RELIEF	
TENS 100 Hz	x
TENS 80Hz	x
FREQ MODULATED TENS	x
ENDORPHINIC	x
DECONTRACTION	x
PULSE WIDTH MOD TENS	x
BURST TENS	x
MIXED TENS	x
SPECIFIC TREATMENT	
DIRECT CURRENT	
IONTOPHORESIS	
HYPERHIDROSIS	
DENERVATED	
TOTAL MANUAL	
PARTIAL MANUAL	
AESTHETIC	
TONING	x
FIRMING	x
INCONTINENCE	
STRESS INCONTINENCE	
URGE INCONTINANCE	
MIXED INCONTINANCE	
POST PARTUM PREV	

CONDITIONING	
STRENGTH	x
EXPLOSIVE STRENGTH	x
HYPERTROPHY	x
PROGRAMMING	
NMES	
CONT CONVENTIONAL	x
CONT MODULATED PW	x
CONTI MODULATED FREQ	x
BURST	x
INTERMITT CONVENTIONAL	x
INTERMITT MOD FREQ	x
TENS	
CONVENTIONNAL	x
BURST	x
MIXED FREQUENCY	x
MODULATED PW	x
MODULATED FREQUENCY	x
2+2	
FREQ MODULATED TENS	
PULSE WIDTH MOD TENS	
BURST TENS	
ENDORPHINIC	
REINFORCEMENT (SMALL)	
REINFORCEMENT (LARGE)	
DISUSE ATROPHY (SMALL)	
DISUSE ATROPHY (LARGE)	

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