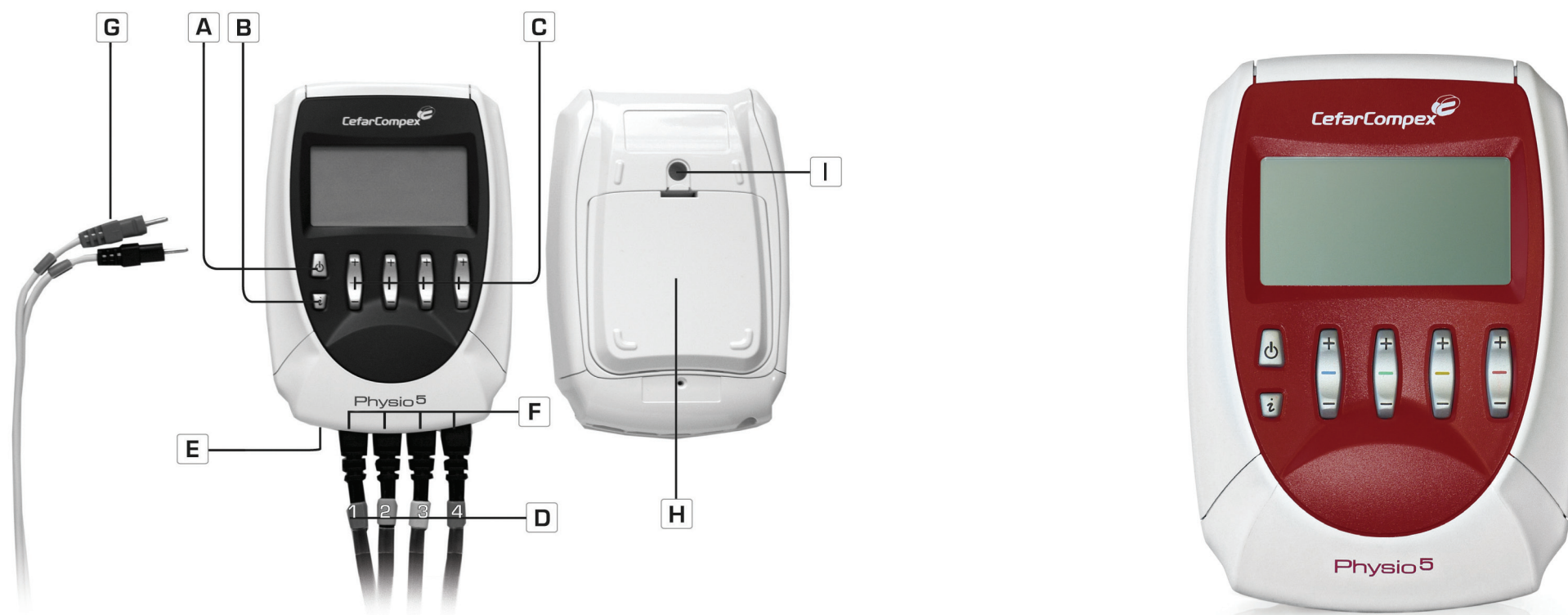
Physio 5 Quick Start Guide

M321317 v.1

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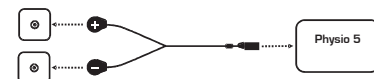
Description of stimulator

- A - On/Off button
 B - i button. This is used to:
 • Increase stimulation energies for several channels simultaneously.
 • Access the Top 5 menu (showing 5 most recently used programmes).
 • Access parameters info menu for personalised programmes.
 C - +/- buttons for 4 stimulation channels
 D - Sockets for 4 stimulation cables
 E - Battery charger socket
 F - Stimulation cables Channel 1 = blue Channel 2 = green Channel 3 = yellow Channel 4 = red
 G - Stimulation cable fitted with pin connector
 H - Rechargeable battery compartment
 I - Belt clip socket

1. Connecting the cables

2. Selecting language, contrast and volume

- b: Use the channel 1 +/- button to choose the language you wish to use.
 c: Use the channel 2 +/- button to adjust the contrast of the screen.
 d: Use the channel 3 +/- button to adjust the volume.
 e: Use the channel 4 +/- button to adjust the backlighting.
 On: Backlighting always on.
 Off: Backlighting always off.
 Auto: Backlighting activated whenever a button is pressed.
 a: Use the On/Off button to confirm and save your choices. The settings will be applied immediately.

3. Choosing a type of treatment

- a: Press the On/Off button to turn off the unit.
 b: Use the channel 1 +/- button to choose another type of treatment (i.e.: Specific Treatment or Conditioning).
 c: Press the channel 2 +/- button to confirm your choice and access the programme category selection screen.

4. Choosing a programme category

- a: Press the On/Off button to return to the previous screen.
 b: Use the channel 1 +/- button to choose a category.
 c: Press the channel 4 +/- button to confirm your choice and access the programme selection screen.

5. Choosing a programme

- a: Press the On/Off button to return to the previous screen.
 b: Use the channel 1 +/- button to choose the programme you want.
 c: Press the channel 4 +/- button to confirm your choice. Certain programmes start right away while others allow you to specify additional options.

6. Personalising a programme

- a: Press the On/Off button to return to the previous screen.
 b: Use the channel 1 +/- button to choose the desired treatment area. You can choose from 7 areas.
 c: Press the channel 2 + button to delete the warming-up sequence and 2 - to choose if you want to combine with any of the 2+2 programmes on channel 3 and 4.
 d: Use the channel 3 +/- button to choose the programme level.
 e: Use the channel 4 +/- button to confirm your choices and start the programme.

7. Starting a programme

- a: Press the On/Off button to interrupt the programme momentarily.
 b c d e: The unit will beep and the symbols for the 4 channels will alternate between + and 000. The energy level for the 4 channels is at 0. To start the programme, you will need to increase the energy levels in the channels you are going to use. To do this, use the corresponding + buttons. The different energies reached during the contraction phase are shown by a series of black bar graphs. Active rest phase energies are shown by hatched bar graphs.

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