USERS MANUAL

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MANUAL DO UTILIZADOR

MANUALE D'USO

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BRUKSANVISNING

KÄYTTÖOHJE

cefar[®] easy

1

2

3

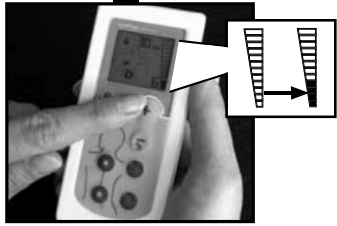
4

5

6

2. MEDICAL BACKGROUND

WHAT IS TENS?

TENS = Transcutaneous Electrical Nerve Stimulation. TENS is an effective method offering Pain Relief. TENS utilises the nervous system's own pain relief mechanisms to ease both acute and chronic pain.

WHEN IS TENS USED?

Good results are observed in acute and long-term pain conditions of many kinds, but above all when the pain originates in joints, the skeleton, muscles, skin, viscera, or nervous system. In some cases, TENS on its own may be sufficient, and in other cases it may provide a valuable complement to other forms of treatment.

HOW DOES IT WORK?

With the Pain Relief programs, impulses are sent through the nerves to block the pain impulses. Pain Relief is usually most efficient during stimulation, but the effect can also last after the program has finished. It also increases the circulation. The Massage programs increases the circulation and decreases the tension in the muscles.

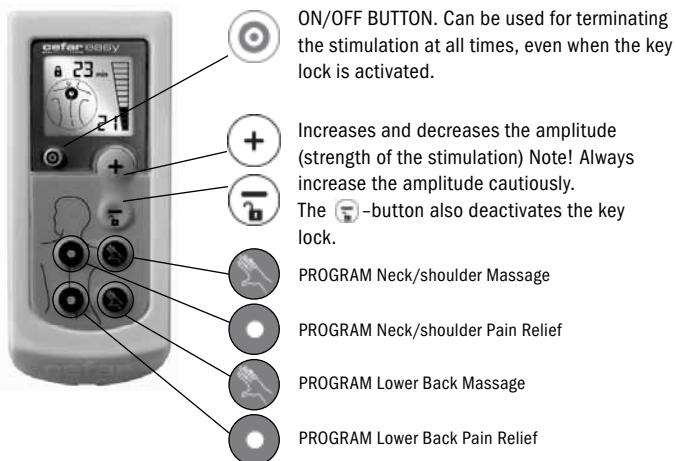
The Pain Relief programs do not always treat the cause of pain. If the pain persists, seek medical advice.

WHEN SHOULD I USE TENS?

The programs in the CEFAR EASY can be used on any occasion when Pain Relief or Muscle Relaxation is needed. All the programs can be used as often as required. Every session should last at least a minimum of 30 minutes up to several hours.

4. OVERVIEW OF THE CEFAR EASY

DISPLAY SYMBOLS

Program selected:
Neck/Shoulder Pain Relief

Program selected:
Neck/Shoulder Massage

Program selected:
Lower Back Massage

Program selected:
Lower Back Pain Relief

AUTOMATIC KEY LOCK is activated. The key lock is activated 10 seconds after the amplitude has been set. Deactivate by pressing .

BATTERY CHANGE NEEDED. When you see this symbol in the display, it means that the batteries very soon will run out.

Broken circuit symbol. See section 9.

AMPLITUDE. Ongoing amplitude is shown in both digits and as an amplitude bar. The amplitude bar changes to the next square after 5 mA. Lower the amplitude to 0 mA to pause the program.

Indicates how much is left of the program to run.

6. REPLACEMENT OF BATTERIES

When you see the battery symbol  in the display, it means that the batteries very soon will run out. As long as stimulation feels the same as usual you can continue the treatment. It is time to replace the batteries when stimulation starts diminishing or the stimulator stops working.

-
- Switch off the stimulator.
 - Remove the battery cover, take out the batteries.
 - Make sure that the new batteries are placed correctly with respect to the polarity (+ and - symbols). See picture.
 - Dispose the used old batteries at a recycling station.

Rechargeable batteries should be charged when the indicator lights up or the stimulator stops working. **NOTE!** Never try charging non-rechargeable batteries, due to the risk of explosion.

If the CEFAR EASY is not used for some time (approximately 3 months), the non-rechargeable batteries should be removed from the stimulator.

7. CARE INSTRUCTIONS

Taking care of and cleaning the CEFAR EASY equipment is simple, provided the following instructions are observed:

- Never expose the stimulator to water. Wipe it off with a damp cloth if necessary.
- The cable is best preserved if left attached to the stimulator between sessions.
- Do not jerk the cable or connection.
- The stimulator does not need any other maintenance or calibration.

8. ACCESSORIES

Cefar also offers accessories to the CEFAR EASY.

For purchase information, contact your Cefar dealer or visit www.cefar.se.

- **CEFAR EASY BELT**

A lower-back belt is available as an accessory to the CEFAR EASY. The CEFAR EASY belt makes it easier to relieve lower-back pain quickly. The belt is designed to give you warming support and correct placement – every time!

- **CEFAR EASY SELF-ADHESIVE ELECTRODES**

Electrodes get worn out and need to be replaced. It is recommended to replace the electrodes after approximately 20-40 times of usage.

- **Y-CABLE**

With the help of a so-called Y-cable, the number of electrodes can be doubled.

9. TROUBLESHOOTING

THE STIMULATION DOES NOT FEEL THE SAME AS USUAL

- Check that all settings are correct and make sure that the electrodes are correctly placed.
- Slightly change the position of the electrodes.

THE STIMULATION CURRENT FEELS UNPLEASANT

- The skin is irritated.
- The electrodes begin to lose their stickiness and do not stick properly to the skin. Moisten the adhesive surface with a few drops of water before placing on the skin.
- The electrodes are worn out and need to be replaced.

- Slightly change the position of the electrodes.

THE STIMULATION FEELS WEAK OR NOT AT ALL

- Check if the batteries need to be replaced.
- Electrodes are too old and need replacement.

THE BROKEN CIRCUIT SYMBOL IS SHOWN ON THE DISPLAY SYMBOL

The broken circuit symbol indicates that the resistance is too high, or that a cable is broken.

- A too high resistance can be caused by a bad connection between the electrodes and your skin, or that the electrodes need to be replaced.
- A cable breakage can be checked by pressing the cable's pins against one another while increasing the amplitude for the corresponding channel to 11 mA. If the amplitude now drops to 0.0 mA and  starts flashing, the cable needs to be replaced.

Note! Never increase the amplitude above 20 mA when you check for cable breaks, since this can damage the stimulator.

THE STIMULATOR IS NOT WORKING

If the display to the left appears when you start the stimulator, it means that the stimulator is broken and needs to be replaced. **NOTE!** Do not use the stimulator - contact your Cefar dealer.

10. TECHNICAL DATA

CEFAR EASY is an electrical stimulator intended for Pain Relief and Massage. It delivers a constant current of 60 mA through a load of 1000 Ω .

INFORMATION RELATED TO ELECTROMAGNETIC COMPATIBILITY (EMC).

CEFAR EASY is designed to be used in typical domestic or commercial environments and approved according to the EMC safety standard of EN 60601-1-2.

CEFAR EASY emits very low levels in the radio frequency (RF) interval. Therefore it is not likely to cause any interference in nearby electronic equipment (radios, computers, telephones etc.).

CEFAR EASY is designed to withstand foreseeable disturbances originating from electrostatic discharges, mains supply magnetic fields and radio frequency transmitters (such as mobile telephones).

12. PRECAUTIONARY MEASURES

- **Warning!** Users with implanted electronic equipment, such as pacemakers and intracardiac defibrillators, must not use the CEFAR EASY. Pregnant women should not use the CEFAR EASY during the first trimester, and should always consult a doctor, midwife or physiotherapist prior to use.
- Do not stimulate on the front or sides of the neck, since a drop in blood pressure can occur.
- Do not use the CEFAR EASY if you are connected to, or in the vicinity of, high-frequency surgical or industrial equipment. This may cause burn injuries on the skin under the electrodes, as well as problems with the stimulator.
- Observe caution when using the CEFAR EASY in the immediate vicinity of cellular phones that are switched on.
- Do not use the CEFAR EASY in the direct vicinity of short-wave or microwave equipment, since this may affect the output power of the stimulator.
- Keep out of reach from children.
- The stimulator should only be used with skin electrodes intended for nerve and muscle stimulation.
- The electrodes should only be placed on healthy skin. Avoid skin irritation by ensuring good contact between electrodes and skin. Hypersensitivity can occur in rare cases. If skin irritation should occur, treatment with the CEFAR EASY should be temporarily discontinued. If condition persists, seek medical advice.
- Switch off stimulation before removing the electrodes from the skin. If an electrode comes off, shut off the stimulation before picking it up. Getting electrical stimulation through the fingers is unpleasant but not harmful.
- Exercise caution when using electrodes smaller than 5 cm², as the current density in some cases exceeds 2 mA rms/cm² and increases risk for burn injuries.
- Be careful with stimulating in the heart region.
- CEFAR Medical AB does not take any responsibility for any electrode placements other than recommended.
- Muscle soreness may occur after stimulation with CEFAR EASY, but usually disappears within a week.
- Only Cefar accessories should be used with the CEFAR EASY.
- Use the CEFAR EASY only as stated in the operating instructions.
- Inspect the equipment prior to use.
- Never open the battery lid during stimulation.
- This stimulator meets the requirements of MDD 93/42/EEC, including the EMC requirements. This, however, is no guarantee that the equipment will not be affected by electromagnetic disturbances.
- Observe caution if you use the stimulator while driving, unintentional stimulation changes might extract focus from the driving and create a hazardous situation.

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- CEFAR Medical AB ei ole vastuussa, jos elektrodit on kiinnitetty suositusten vastaisesti.
CefarCompex Scandinavia AB | Murmansgatan 126 | SE-212 25 Malmö | Sweden